

WARNINGS

When transporting the cart, it should be pushed and not pulled.

To prevent tipping, take caution when moving over an elevator threshold, expansion joint or other change in floor level.

Do not move the cart faster than a normal walking pace.

If using an electronic lock, it is strongly recommended that batteries be replaced annually.



Load heavy items at the bottom of the cart and not at the top. Do not exceed stated load limits

Do not lift the cart by its handle.

Do not force hinged doors to side of cart


Take caution when pulling out trays/ baskets to not accidentally pull it out of the cabinet.

Do not use in a cart wash, or other high moisture environment

USERS GUIDE

CASTERS

The two front casters have brakes. The brakes should be set when working at the cart. To activate the brake, step down on the pedal. To deactivate the brake, lift-up on the pedal. The two rear casters are simple swivel casters for easier transport.

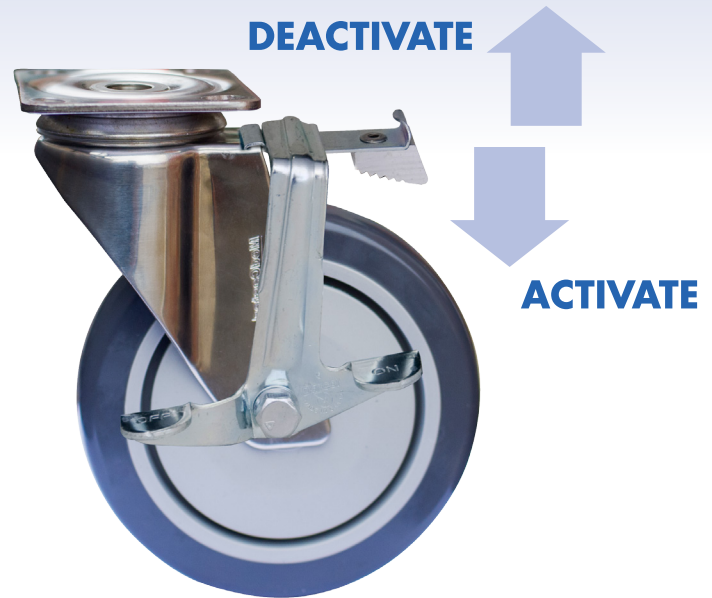

A



B

LOCKS

Key – With a tambour door unit, the lock is on right side. (Figure A)

Keyless Lock – refer to the CodeLock Kitlock manual found in a box in each unit. (Figure B)

ACCESSORIES

LogiQuip® baskets have built-in stops to minimize the risk of accidentally pulling them out of the cart.

LogiQuip® has been in the healthcare storage and distribution systems business since 1992. Our value-added services support the hospital clinician, logistics officer, purchaser and planner.

Because each hospital's inventory, supply, and storage requirements are diverse, our mission is to solve your specific storage and distribution issues. With "space" being one of the hospital's most expensive capital assets, our philosophy is to maximize storage efficiency while supporting infection control and functional ergonomics. Whether a single cart or a complete supply storage system, our experienced healthcare storage solutions adviser can offer a solution to your challenges.