



ASSEMBLY GUIDE



LQS WRAP RACK



CAUTION

Read and understand all of the instructions and safety information in this document before assembling this product.

SAVE THESE INSTRUCTIONS


LQS WRAP RACK PARTS & HARDWARE

TOOLS NEED

- Wood Block
- Nylon or Rubber Base Hammer

PARTS & HARDWARE	QUANTITY FOR 48" LENGTH UNIT	QUANTITY FOR 60" LENGTH UNIT
1. Post	4	4
2. Shelf	1	1
3. Slant Rail Set (R,L)	1	1
4. 3-Sided Support Frame	2	2
5. Split Sleeve (1 Pair)	16	16
6. Wrap Bar*	4	4
7. O-Rings	20	20
8. 5" Casters, Swivel w/Brake	2	2
9. 5" Casters, Swivel	2	2
10. Donut Bumper	4	4
11. Dust Shelf Cover	1	1

*Each Wrap Bar comes with five (5) Retaining Rings

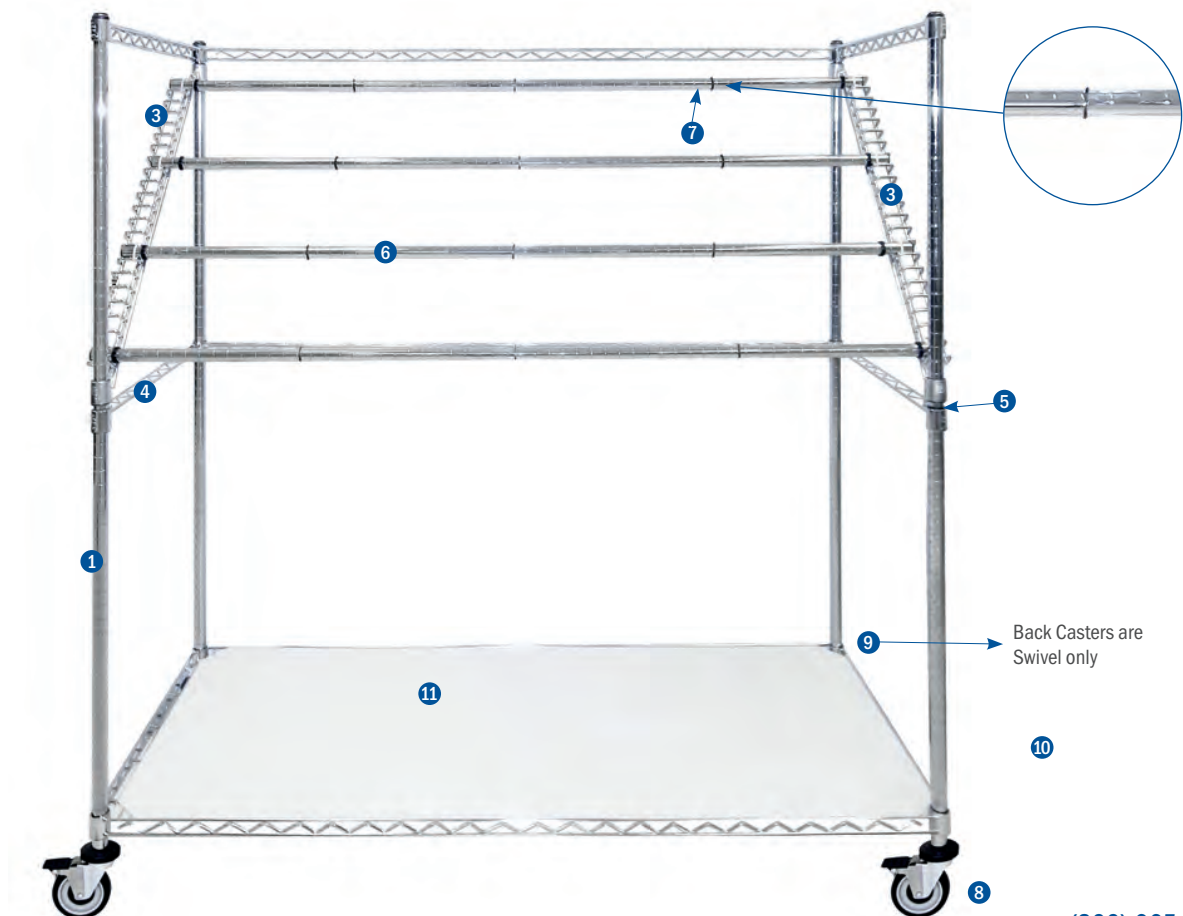

LQS WRAP RACK ASSEMBLY INSTRUCTIONS

1. CASTERS, SHELF AND DUST SHELF COVER

Insert the two (2) swivel with brake casters into the two front posts. Insert the two (2) swivel only casters into the two back posts. Assemble split sleeves on the first groove found on the bottom post. The split sleeves snap together around the posts. The wider end of the split sleeve faces the bottom of the assembly. Lower the shelf down on the posts until the split sleeves rest snugly in the shelf collars. Place the dust shelf cover on the bottom shelf.

2. BOTTOM THREE-SIDED FRAME

Assemble the split sleeves in the middle of the posts (approximately 15"-20" above the bottom groove). Slide the three-sided frame down the posts until it fits securely over the split sleeves.

3. SLANT RAILS


Install one pair of split sleeves into the desired groove location (2"-3" above the three-sided frame) on each front post. Next install one pair of split sleeves onto each rear post using the 24th groove above the split sleeves on the front posts. Slide the left and right slant rails down the posts until they fit securely over the sleeves.

4. WRAP BAR AND O-RINGS


Slide five (5) o-rings onto each wrap bar, spaced evenly apart. Place all four (4) wrap bars into the slant rails.

5. TOP THREE-SIDED FRAME


Assemble the split sleeves on the top of the posts. Slide the three-sided frame down the posts until it fits securely over the split sleeves.

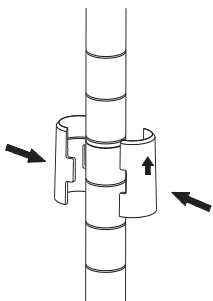

Snap plastic tapered split sleeves on each post

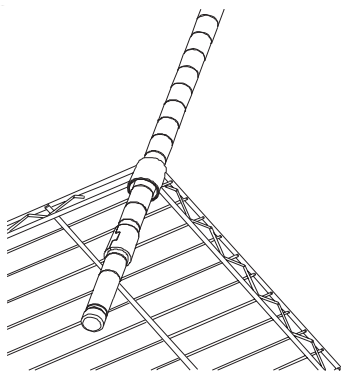

Install shelf by sliding over the posts

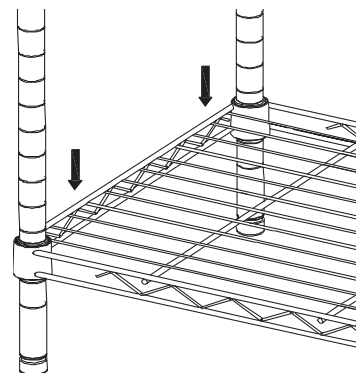

Tap very lightly with a nylon or rubber base hammer to seat the shelf on the split sleeves. To prevent damaging the shelf you may use a wood block to evenly disperse the force from the nylon/rubber based hammer



LIMITED 5-YEAR WARRANTY

LogiQuip products are covered by a Limited 5-Year Warranty from date of receipt against defect in materials or workmanship under normal use and service, excluding Electronic Components, Cart Covers, Custom Products, LogiCell Tambour Doors, Label Holders, and Casters. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. Visit [LogiQuip.com/Warranty](https://www.logiquip.com/Warranty) for more details.

CUSTOMER SERVICE

If you have any problems with your LQS Wrap Wrack, please contact our Customer Service Department at:

PHONE: [\(800\) 665-3760](tel:8006653760)

EMAIL: CustomerSupport@LogiQuip.net

ABOUT US

Healthcare exclusive since 1992, LogiQuip's Inventory Management Solutions are designed for Healthcare Professionals looking for a better way to manage supply chain tasks. With Expert Advice, Innovative Solutions, and 360 Service that doesn't stop after the sale, LogiQuip's sole purpose is to ensure Healthcare Professionals have the supplies they need when they need them. Partnering with you to design and implement the right solutions, we give you time back to focus on what's important; Patient Care.