



## INSTALLATION GUIDE

# Dri-Stor CHANNEL PURGE UPGRADE



Read and understand all of the instructions and safety information in this document before operating this product.

**SAVE THESE INSTRUCTIONS**

## STEP 1:

For side compartment models, take top off of the main scope cabinet to access the attic. There will be eight (8) Philips screws. For top compartment models take the top off.



## STEP 2:

Take out the black plugs covering holes that are already punched into the ceiling of the cabinet.

## STEP 3:

Add white quick connect plugs from the bottom and secure with threaded nut from above. Make sure the silver button is facing forward.



## STEP 4:

Tighten the nut down from above to secure the quick connect.


## STEP 5:

For side compartment cabinets, secure the black manifold to the back of the cabinet with the large barb facing to the side compartment with the bolts and washers that are supplied.



## STEP 6:

For top mount cabinets, place the manifold, with the large barb facing to the right and secure with double-sided tape.

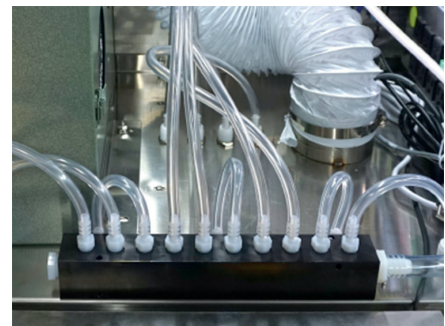

## STEP 7:

From inside the side compartment, push the ½" tubing through the back hole and connect to the large barb on the manifold.




Back Hole



Large Barb Manifold

## STEP 8:

Take blue pump and add rubber elbow to the side. Secure with the butterfly clip.

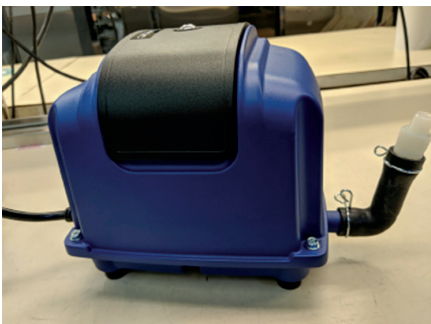

## STEP 9:

Screw white HEPA filter onto top of elbow and place on second shelf of side cabinet or in top of top-mount cabinet. Place ½" tubing on the barb on top of the HEPA filter.



## STEP 10:

On the side compartment cabinet, you will find two plugs beside the power switch. Take those out.



## STEP 11:

Add the red and yellow LED lights, putting an O-ring on, and then push the wire through the hole. Put nut on the wire and then secure to the cabinet.



## STEP 12:

Once LED's are installed connect ends to the gray wire.



## STEP 14:

Once connected turn knurled knob to secure.



## STEP 13:

When wires are connected, thread the barrel connector end to the back of the control box. Make sure the male pins in the box line up when connecting the cable.



## STEP 15:

Plug control box into power bar and plug the blue channel purge pump into Pump 1 on the box. The cabinet is now ready to turn on. After turning on, push the white button on the box to reset filter change timer. Put the side compartment door on and the cabinet is ready to use for channel purge.



## LIMITED 5-YEAR WARRANTY

LogiQuip products are covered by a Limited 5-Year Warranty from date of receipt against defect in materials or workmanship under normal use and service, excluding Electronic Components, Cart Covers, Custom Products, and LogiCell Tambour Doors. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. Visit [LogiQuip.com/Warranty](https://www.logiquip.com/Warranty) for more details.