



Dri-Stor

CABINETS



WHY LOGIQUIP®?



Healthcare exclusive since 1992, LogiQuip's **Inventory Management Solutions** are designed for Healthcare Professionals looking for a better way to manage supply chain tasks. With **Expert Advice**, **Innovative Solutions**, and **360° Service** that doesn't stop after the sale, LogiQuip's sole purpose is to ensure Healthcare Professionals have the supplies they need when they need them. Partnering with you to design and implement the right solutions, we give you time back to focus on what's important, Patient Care.

EXPERT ADVICE

With an in-depth understanding of the industry and a focus on **Infection Control** and **Lean Processes**, we partner with you to ensure we design the right solutions for you.

INNOVATIVE SOLUTIONS

Designed with an eye on the horizon, LogiQuip has **Innovative Solutions** that evolve with your facility. Our flexible and efficient options help to ensure you have the supplies you need when you need them.

360° SERVICE

LogiQuip's experienced **Sales Representatives** and **Award-Winning Customer Support Team** provide unparalleled service before, during, and after the sale.



LET US DO THE HEAVY LIFTING

WITH LABOR SHORTAGES AND THE EVER-INCREASING DEMAND TO DO MORE WITH LESS, the **LogiQuip Services Team** can help to decrease that burden with turnkey solutions to help ensure you have the supplies you need when you need them.

PROJECT MANAGEMENT

From our initial meeting with you through implementation, a LogiQuip **Services Team Member** will be assigned as your **Project Manager**, working with you every step of the way. Your **Project Manager** will serve as your single point of contact during the project life cycle. Together we will develop and implement project milestones until the work is complete. Contact your **Sales Representative** or call **(800) 665-3760** for more information.

ON-SITE ANALYSIS

Working with your LogiQuip **Representative**, an on-site analysis will be done to ensure that the **Project Manager** has a complete understanding of your current environment. During our on-site analysis we will review your current state, identify gaps, and align on the goals of the project.

PLANNING

Using a mix of innovative products, Lean principles, tried and true methods, and out of the box thinking, LogiQuip will provide a holistic plan with turnkey solutions to minimize downtime while decreasing your storage footprint and increasing PAR levels.

INSTALLATION & CONVERSION

LogiQuip is dedicated to providing on-time delivery, efficient installation, and supply conversion to minimize down time and time on site to get you up and running as quickly as possible.

ONGOING SUPPORT

With the project complete, we want to help you sustain the changes that have been made. With **Annual Inventory Reviews**, **Preventative Maintenance Plans**, and **Field Service** for in warranty and out of warranty repairs, we have you covered to keep things running smoothly.

TABLE OF CONTENTS

TABLE OF CONTENTS

WHY LOGIQUIP®? 2

LOGIQUIP SERVICES TEAM 3

LOGIQUIP DRI-STOR

2021 GUIDELINES. 5

FEATURES & BENEFITS6-7

HEPA POWER VENT8-9

HEPA POWER VENT WITH CHANNEL PURGE10-13

MAINTENANCE PLANS 14

FILTERS & KITS 15



ANSI/AAMI ST91: 2021 COMPLIANT

LogiQuip **Dri-Stor Scope Cabinets** are compliant with the latest **ANSI/AAMI ST91: 2021 Guidelines**, emphasizing the importance of drying the internal channels of flexible and semi-rigid endoscopes. LogiQuip **Channel Purge Dri-Stor Scope Cabinets** forces **HEPA Filtered Air** through **Tubing Channels** connected to individual **Scopes** for 30-minute cycles (30 minutes on/20 minutes off) to aid in thorough drying inside and out.

“ Methods that employ active drying of Endoscopes with filtered air are the preferred means of drying the internal channels of Endoscopes after processing. Storage Cabinets are used to secure and, in some cases, circulate air through the processed Endoscopes. ”

ANSI/AAMI ST91: 2021 Flexible and semi-rigid Endoscope processing in health care facilities, Section 11.1



Several **Channel Purge Dri-Stor** units are **IN STOCK** and ready-to-ship through our **Rapid Ship Program**. Contact LogiQuip for more details.

LOGIQUIP® DRI-STOR FEATURES & BENEFITS

The cost of infection from improperly cleaned and stored endoscopes can be significant. For this reason, LogiQuip developed the **Dri-Stor Line of Scope and Probe Drying Cabinets**. These AORN, SGNA, ANSI, and AAMI Guideline-Compliant **Endoscope Cabinets** start with **304 Grade Stainless Steel** for easy to clean durability. **HEPA Filtered Air** circulates through each **Endoscope** and **Cabinet** to aid in complete drying inside and out. Add **EndoTrack** to control scope access and hang time to help ensure compliance.

304 STAINLESS STEEL CONSTRUCTION

All cabinets start with **304 Stainless Steel** body and **Removable Drip Tray** for easy cleanability, long-lasting durability, and better prevention of microbial growth. **All-welded Joints and Construction** free of fasteners, screws, and rivets provide smooth surfaces throughout.



HOLDS UP TO 20 SCOPES

Each **Cabinet** can hold up to 20 **Scopes** and can accommodate **Large** and **Small Diameter Scopes** as well as **TEE Probes** and **Ultrasound Probes**.

HEPA FILTERED AIR

Our **HEPA Power Vent** and **HEPA Power Vent with Channel Purge** models use **HEPA Filters** that remove **99.97% of particulates 0.3 micron in size** to ensure that the air moving through the **Cabinet** and each **Scope** is clean.



COATED SCOPE HANGERS

Durable, easy-to-clean **Scope Hangers** provide cushioned storage which allows you to hang **Scopes** vertically to help promote drying. Free of sharp edges, each **Scope Hanger** is built with a steel core and coated in resilient, non-porous plastic.

LOGIQUIP® DRI-STOR FEATURES & BENEFITS




CHANNEL PURGE OPTIONS

HEPA Power Vent with Channel Purge technology provides thorough Endoscope drying, inside and out. Forced filtered air passes through tubing channels assigned to individual scopes for 30-minute cycles. Airflow is discontinued for 20 minutes before a new cycle begins. The filtered air quality is 0.3 micron at 99.97% efficiency. Cabinets designed for 11 or more Endoscopes are equipped with two (2) Channel Purge Pumps.

ENDOTRACK SYSTEM

EndoTrack allows you to easily manage and track Endoscope equipment from an intuitive interface. Utilizing Barcode or RFID Technology, the EndoTrack System provides controlled access and security to your Cabinet as well as the ability to track the hang time of each Scope.

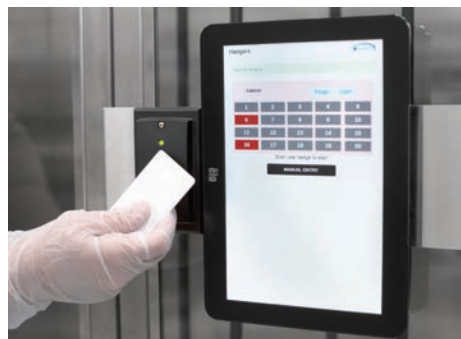

SIDE COMPARTMENT ACCESS

In addition to our Top Compartment Access models, we also have Side Compartment Access models for quick access to interior components for maintenance and expansion.

MODEL #
43690SCHG-20PS (left)
43890SCHG-20PS-ADD-TS (right)

MAINTENANCE PLANS

LogiQuip is committed to customer satisfaction throughout the life of your product. With this in mind, we are pleased to offer Maintenance Plans to keep your new LogiQuip Dri-Stor Drying Cabinet running like new. These hassle-free plans take the maintenance of your Dri-Stor Drying Cabinets off of your to-do list. See page 14 for Maintenance Plans.

LIMITED 5-YEAR WARRANTY

LogiQuip products are covered by a Limited 5-Year Warranty from date of receipt against defect in materials or workmanship under normal use and service, excluding Electronic Components, Cart Covers, Custom Products, and LogiCell® Tambour Doors, Label Holders, and Casters. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. Visit LogiQuip.com/Warranty for more details.

LOGIQUIP® DRI-STOR HEPA POWER VENT

HEPA POWER VENT

Dri-Stor Scope and Probe Cabinets equipped with a **HEPA Power Vent** circulates **HEPA Filtered Air** throughout the **Cabinet** for infection-control conscious drying and storage. The resulting pressurized environment creates a barrier, preventing non-filtered air from entering.

TOP MOUNT — TEE PROBE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
42380SCLG-3T	Holds 3 Tee Probes, Left Hinge Glass Door	18 x 27 x 87
42380SCRG-3T	Holds 3 Tee Probes, Right Hinge Glass Door	18 x 27 x 87
42580SCHG-5T	Holds 5 Tee Probes, Double Glass Doors	18 x 36 x 87

TOP MOUNT — SMALL SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
42380SCRG-6PT	Holds 6 Small Scopes, Right Hinge Glass Door	18 x 27 x 87
42580SCHG-10PT	Holds 10 Small Scopes, Double Glass Doors	18 x 36 x 87

TOP MOUNT — LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
42490SCHG-10PT	Holds 10 Large Scopes, Double Glass Doors	20 x 30 x 95
43490SCHG-10PT	Holds 10 Large Scopes, Double Glass Doors	24 x 30 x 92
42590SCHG-16PT	Holds 16 Large Scopes, Double Glass Doors	20 x 36 x 95
43590SCHG-16PT	Holds 16 Large Scopes, Double Glass Doors	24 x 36 x 92
42690SCHG-20PT	Holds 20 Large Scopes, Double Glass Doors	20 x 44 x 95
43690SCHG-20PT	Holds 20 Large Scopes, Double Glass Doors	24 x 44 x 92



MODEL #
42490SCHG-10PT



MODEL #
43690SCHG-20PT

LOGIQUIP® DRI-STOR HEPA POWER VENT



MODEL #
43790SCHG-16PS



MODEL #
43690SCHG-10PS (left)
43690SCHG-10PS-ADD-TS (right)

SIDE MOUNT – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PS	Holds 10 Large Scopes, Double Glass Doors	24 x 43 x 92
43790SCHG-16PS	Holds 16 Large Scopes, Double Glass Doors	24 x 49 x 92
43890SCHG-20PS	Holds 20 Large Scopes, Double Glass Doors	24 x 57 x 92

ADD-ON – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PS-ADD	Holds 10 Large Scopes, Double Glass Doors, Add-On	24 x 30 x 92
43790SCHG-16PS-ADD	Holds 16 Large Scopes, Double Glass Doors, Add-On	24 x 36 x 92
43890SCHG-20PS-ADD	Holds 20 Large Scopes, Double Glass Doors, Add-On	24 x 44 x 92

SECURE ACCESS – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
43890SCHG-20PS-TSBC	Holds 20 Large Scopes, Double Glass Doors, Bar Code	24 x 57 x 92
43890SCHG-20PS-ADD-TS	Holds 20 Large Scopes, Double Glass Doors, Add-On, For EndoTrack	24 x 44 x 92

LOGIQUIP® DRI-STOR HEPA POWER VENT WITH CHANNEL PURGE

HEPA POWER VENT WITH CHANNEL PURGE

HEPA Power Vent with Channel Purge Technology helps to promote thorough Endoscope drying, inside and out. **Forced filtered** air passes through tubing channels assigned to individual **Scopes** for 30-minute cycles. Airflow is discontinued for 20 minutes before a new cycle begins. The **Filtered Air** quality is to 0.3 micron at 99.97% efficiency. **Cabinets** designed for 11 or more **Endoscopes** are equipped with two (2) **Channel Purge Pumps**.

HEPA CHANNEL PURGE TOP MOUNT – SMALL SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
42380SCLG-6PTD	Holds 6 Small Scopes, Left Hinge Glass Door	18 x 27 x 87
42380SCRG-6PTD	Holds 6 Small Scopes, Right Hinge Glass Door	18 x 27 x 87
42580SCHG-10PTD	Holds 10 Small Scopes, Double Glass Doors	18 x 36 x 87



MODEL #
42580SCHG-10PT

HEPA CHANNEL PURGE TOP MOUNT – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
42490SCHG-10PTD	Holds 10 Large Scopes, Double Glass Doors	20 x 30 x 95
43490SCHG-10PTD	Holds 10 Large Scopes, Double Glass Doors	24 x 30 x 92
42590SCHG-16PTD	Holds 16 Large Scopes, Double Glass Doors	20 x 36 x 95
42690SCHG-20PTD	Holds 20 Large Scopes, Double Glass Doors	24 x 44 x 95
43590SCHG-16PTD	Holds 16 Large Scopes, Double Glass Doors	24 x 36 x 92
43690SCHG-20PTD	Holds 20 Large Scopes, Double Glass Doors	24 x 44 x 92




MODEL #
42490SCHG-10PT

LOGIQUIP® DRI-STOR HEPA POWER VENT WITH CHANNEL PURGE



MODEL #
43890SCHG-20PSD

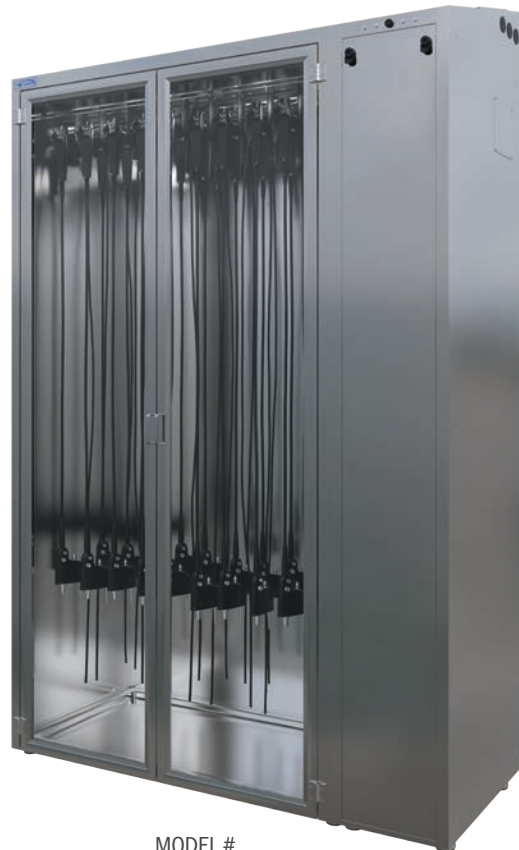

MODEL #
43890SCHG-20PSD

HEPA CHANNEL PURGE SIDE MOUNT – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PSD	Holds 10 Large Scopes, Double Glass Doors	24 x 43 x 92
43790SCHG-16PSD	Holds 16 Large Scopes, Double Glass Doors	24 x 49 x 92
43890SCHG-20PSD	Holds 20 Large Scopes, Double Glass Doors	24 x 57 x 92

HEPA CHANNEL PURGE ADD-ON – LARGE SCOPE CABINETS

MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PSD-ADD	Holds 10 Large Scopes, Double Glass Doors, Add-On	24 x 30 x 92
43790SCHG-16PSD-ADD	Holds 16 Large Scopes, Double Glass Doors, Add-On	24 x 36 x 92
43890SCHG-20PSD-ADD	Holds 20 Large Scopes, Double Glass Doors, Add-On	24 x 44 x 92

LOGIQUIP® DRI-STOR HEPA POWER VENT WITH CHANNEL PURGE



Barcode Reader



RFID Technology

MODEL #
43790SCHG-16PSD-TSBC-TS
with Barcode Reader

HEPA POWER VENT WITH CHANNEL PURGE SECURE ACCESS TOP MOUNT

BAR CODE MODEL	RFID MODEL	DESCRIPTION	D x W x H - IN.
42490SCHG-10PTD-TSBC	42490SCHG-10PTD-TSRF	Holds 10 Large Scopes, Double Glass Doors	20 x 30 x 95
42590SCHG-16PTD-TSBC	42590SCHG-16PTD-TSRF	Holds 16 Large Scopes, Double Glass Doors	20 x 36 x 95
42690SCHG-20PTD-TSBC	42690SCHG-20PTD-TSRF	Holds 20 Large Scopes, Double Glass Doors	20 x 44 x 95
43690SCHG-20PTD-TSBC	43690SCHG-20PTD-TSRF	Holds 20 Large Scopes, Double Glass Doors	24 x 44 x 92

HEPA POWER VENT WITH CHANNEL PURGE SECURE ACCESS SIDE MOUNT

BAR CODE MODEL	RFID MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PSD-TSBC	43690SCHG-10PSD-TSRF	Holds 10 Large Scopes, Double Glass Doors	24 x 43 x 92
43790SCHG-16PSD-TSBC	43790SCHG-16PSD-TSRF	Holds 16 Large Scopes, Double Glass Doors	24 x 49 x 92
43890SCHG-20PSD-TSBC	43890SCHG-20PSD-TSRF	Holds 20 Large Scopes, Double Glass Doors	24 x 57 x 92

HEPA POWER VENT WITH CHANNEL PURGE FOR ENDOTRACK

MODEL	DESCRIPTION	D x W x H - IN.
43690SCHG-10PSD-ADD-TS	Holds 10 Large Scopes, Double Glass Doors, Add-On, For EndoTrack	24 x 30 x 92
43890SCHG-20PSD-ADD-TS	Holds 20 Large Scopes, Double Glass Doors, Add-On, For EndoTrack	24 x 44 x 92

LOGIQUIP® DRI-STOR HEPA POWER VENT WITH CHANNEL PURGE



Scan QR Code for
product details.



LOGIQUIP® DRI-STOR MAINTENANCE PLANS

LogiQuip is committed to customer satisfaction throughout the life of your products. With this in mind, we are pleased to offer two (2) **Maintenance Plans** to keep your new LogiQuip **Dri-Stor Drying Cabinet** running like new. These hassle-free plans take the maintenance of your **Dri-Stor Drying Cabinets** off of your to-do list. Ask your Customer Representative about pricing for the **2- and 4-Year Maintenance Plans**.

STANDARD

WARRANTY

LogiQuip Dri-Stor Cabinets are covered by a **5-Year Limited Warranty** from date of receipt against defect in materials or workmanship under normal use and service, excluding electronic components. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. If the product is replaced, the new product will be warranted for the remaining term of the original product warranty period.

1-Year Warranty on electronic components and sub-assemblies.

2-YEAR

MAINTENANCE + STANDARD WARRANTY

MODEL # LQDS2MP **5-YEAR LIMITED WARRANTY + 24 MONTHS OF MAINTENANCE**

- Maintenance Visits Every 6 Months
- Replacement and Consumable Parts
(1) Main Cabinet Filter (40000MFF)
- Inspect Tubing
- Check Operation of Fans and Pumps
- Check Doors and Locks
- Check Levelness of Cabinets

4-YEAR

MAINTENANCE + STANDARD WARRANTY

MODEL # LQDS4MP **5-YEAR LIMITED WARRANTY + 48 MONTHS OF MAINTENANCE**

- Maintenance Visits Every 6 Months
- Replacement and Consumable Parts
(2) Main Cabinet Filter (40000MFF)
- Inspect Tubing
- Check Operation of Fans and Pumps
- Check Doors and Locks
- Check Levelness of Cabinets

MAINTENANCE SCHEDULE

TYPE OF SERVICE	MODEL NUMBER	6 MONTHS	12 MONTHS
Cabinet Pre-Filter	40000EF	Replace	Replace
Channel Purge HEPA Filter Cannister (if equipped)	40000CPHF	Replace	Replace
HEPA Pre- and Post-Filters (including O-Rings)	40000PPFK	Replace	Replace
Scope Cabinet HEPA Filter*	40000MFF	Replace Every 2 years	
Tubing†	Model # Varies	Inspect	Inspect

*Can be ordered separately

†Tubing replaced is an additional cost


LOGIQUIP® DRI-STOR FILTERS & KITS

ANNUAL PM FILTER KIT

MODEL 40000PPFK

(2) HEPA Fan Pre-Filters, (2) HEPA Post-Filters, 2 O-Rings

Replacement every six months is recommended.

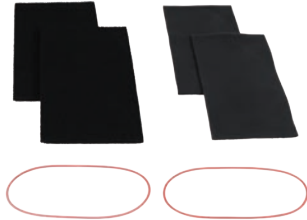

ANNUAL FILTER KIT FOR HEPA-ONLY CABINETS

MODEL 40000HEPAKIT

(2) HEPA Pre-Filters, (2) HEPA Post-Filters, (2) O-Rings,
(2) 40000EF (External Filter)

Replacement every six months is recommended.



ANNUAL FILTER KIT FOR 1 PUMP CABINETS (FOR CABINETS WITH 10 SCOPES OR LESS)

MODEL 40000CPKIT-1

(2) HEPA Pre-Filters, (2) HEPA Post-Filters, (2) O-Rings,
(2) 40000EF (External Filters), (2) 40000CPHF

Replacement every six months is recommended.



ANNUAL FILTER KIT FOR 2 PUMP CABINETS (FOR CABINETS WITH 11 SCOPES OR MORE)

MODEL 40000CPKIT-2

(2) HEPA Pre-Filters, (2) HEPA Post-Filters, (2) O-Rings,
(2) 40000EF (External Filters), (2) 40000EF (External
Filter), (4) 40000CPHF

Replacement every six months is recommended.



SQUARE EXTERNAL FILTER FOR ENDOSCOPE CABINET

MODEL 40000EF

(1) White Square Fiber Filter with Frame

Replacement every six months is recommended.



REPLACEMENT MAIN HEPA FILTER

MODEL 40000MFF

(1) Main HEPA Filter Replacement

Replacement every two years is recommended.



IN-LINE FILTER PHD-02B

MODEL 40000CPHF

(1) Plastic Ball-Shaped Filter

Replacement every six months is recommended.



TUBING

MODEL # VARIES

Tubing Replacement is an Additional Cost

Inspection every six months is recommended.





www.logiquip.com

(800) 665-3760 | customersupport@logiquip.net

5.2023