



INSTALLATION GUIDE



ParStor[®]

INSTALLATION GUIDE



CAUTION



Read and understand all of the instructions and safety information in this document before assembling and installing this product.

PARSTOR FRAME PARTS & HARDWARE

TOOLS NEEDED

- $\frac{3}{8}$ " Socket & Wrench
- $\frac{3}{8}$ " Open End Wrench
- Rubber Mallet

PARTS & HARDWARE	MODEL NUMBER	USE	QTY
1. Frame	FR68-VHD	Structural Support	2
2. Cross Bar	CB18JB-V	Structural Support	4
3. Narrow Cross Bar	CB16JB-V	Structural Support For TU1420RPS	4
4. Slides	SLV, SLT-V	Drawer Extension Slides	2
5. Leveling Foot	PSLF	Leveling	4
6. Leveling Foot Insert	Part of PSLF	Leveling	2
7. Caster - Swivel	5C-DS	Mobile	2
8. Caster - Swivel/Brake	5CB-DS	Mobile	2

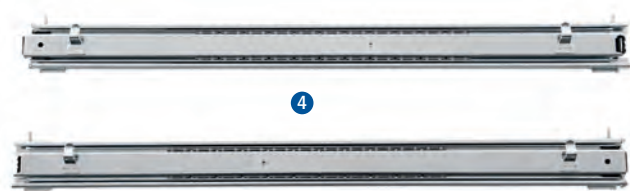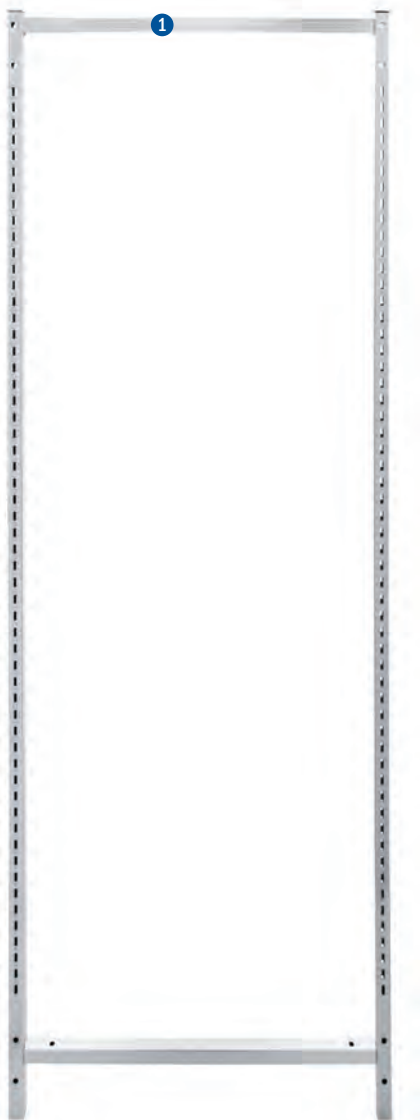

INSTALLATION INSTRUCTIONS

Step 1A - Install Casters (PART 7 and 8). If units are stationary proceed to Step 1B

1. Install the caster into the caster insert with a rubber mallet (Fig. 1)

Step 1B - Install Leveling Feet

1. Using the mallet, pound in the leveling foot insert (PART 6) (Fig. 2)
2. Thread the leveling foot (PART 5) into the insert (Fig. 3)

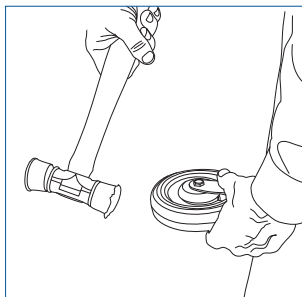


Fig. 1

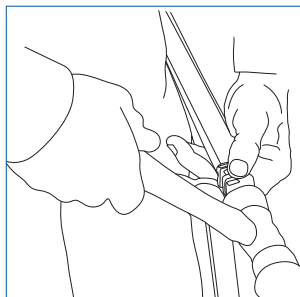


Fig. 2

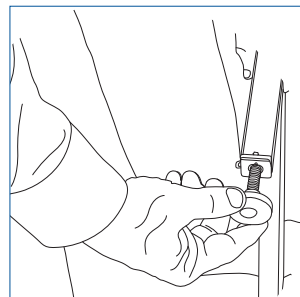


Fig. 3

Step 2 - Frame Construction

1. Using the provided nuts and bolts, attach the cross bars (PART 2 or 3) to the top and bottom of one Frame (PART 1). Assemble all four (4) cross bars to the frames (Fig.4)
2. After you secure the cross bar to the first frame, repeat the same steps on the additional frame. LogiQuip recommends you only connect a maximum of four frames as a single unit (three complete modules).
3. After all cross bars are attached to the frames stand the ParStor® unit upright

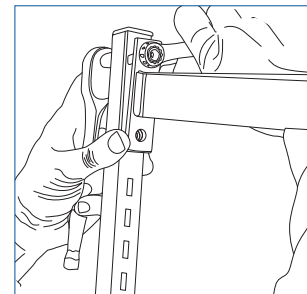


Fig. 4

Step 3 - Assembly Accessories

1. Attached the bottom shelf liner or "insert" to the cross bars by using the provided glue dots (Fig.5)
2. Attach the top shelf to the top of the frame. There are pegs underneath the top shelf that sit in side slots atop the frame (Fig. 6)

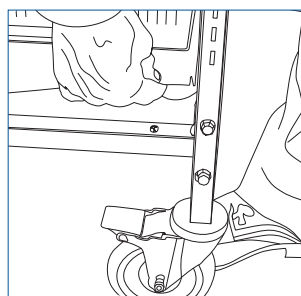


Fig. 5

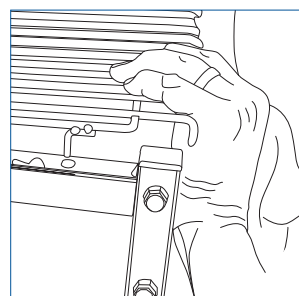


Fig. 6

INSTALLATION INSTRUCTIONS (CONTINUED)

Step 3 - Assembly Accessories (continued)

3. Full-extension slide attach baskets and shelves to our frames. Attach the stand slides (**PART 2**) to the frame in pairs, one level at a time starting with your lowest desired level. To attach a slide, hold the side at an angle and insert the top tab (Fig. 7). Rotate the slide and insert the bottom tab (Fig. 8). the slide will then slightly drop down.

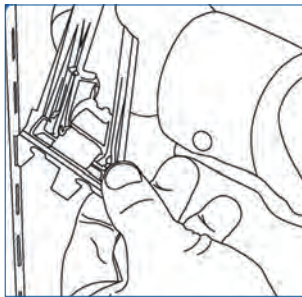


Fig. 7

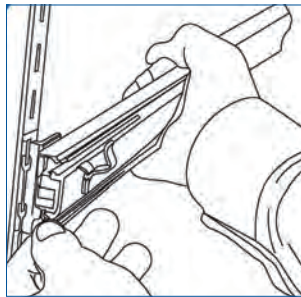


Fig. 8

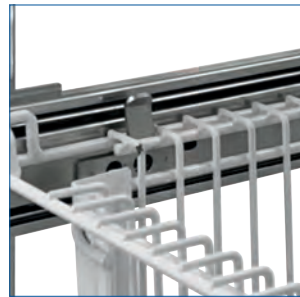


Fig. 9

4. Set the wire shelf or wire baskets on the slides using the hooks on the slides (Fig. 9)
5. To attach plastic tubs to slides use the slide frame. Place the frame on the slide just like the wire baskets and shelves (Fig. 10). Make sure you have the frame in the right position; the smaller frame brace is facing forward.
6. Place the tub inside of the slide frame (Fig. 11)



Fig. 10



Fig. 11

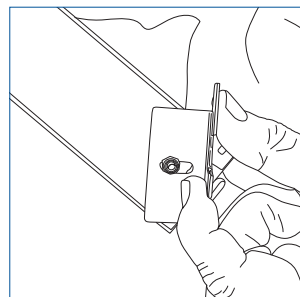


Fig. 12

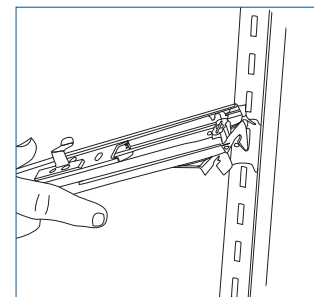


Fig. 13

7. A slanted slide has swiveling bracket tabs (Fig. 12) to allow the front and back of the slide to be placed at different levels. To attach the slanted slide one level at a time starting at your lowest desired level. To attach a slanted slide hold the slide at an angle and insert the top tab (Fig. 13). Rotate the slanted slide and insert the bottom tab (Fig. 14)


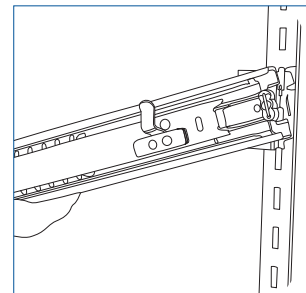


Fig. 14

INSTALLATION INSTRUCTIONS (CONTINUED)

Step 3 - Assembly Accessories (continued)

8. Slanted Slides come with a white safety stop (Fig. 15). Pull the safety stop down to the locked position to prevent slanted baskets from unintentionally sliding out.
9. Insert dividers into the baskets or tubs. To do this correctly, place the long dividers (Fig. 16) in the basket first. Make sure the prongs on the dividers point up. Next, insert the short dividers (Fig. 17). Unlike long dividers, the prongs on the short dividers face down into the basket.

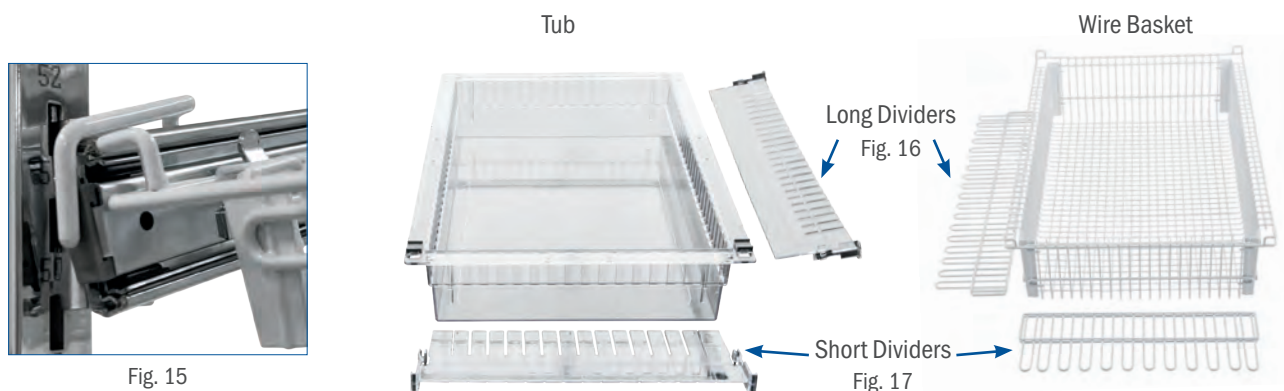

The versatility of this product allows you to put together baskets, tubs, shelves, and dividers to create optimal storage specific to your needs.

LIMITED 5-YEAR WARRANTY

LogiQuip products are covered by a Limited 5-Year Warranty from date of receipt against defect in materials or workmanship under normal use and service, excluding Electronic Components, Cart Covers, Custom Products, and LogiCell Tambour Doors. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. Visit [LogiQuip.com/Warranty](https://www.logiquip.com/Warranty) for more details.

CUSTOMER SERVICE

If you have any problems with your ParStor Cart, please contact our Customer Service Department at:

PHONE: (800) 665-3760 EMAIL: CustomerSupport@LogiQuip.net

ABOUT US

Healthcare exclusive since 1992, LogiQuip's Inventory Management Solutions are designed for Healthcare Professionals looking for a better way to manage supply chain tasks. With Expert Advice, Innovative Solutions, and 360 Service that doesn't stop after the sale, LogiQuip's sole purpose is to ensure Healthcare Professionals have the supplies they need when they need them. Partnering with you to design and implement the right solutions, we give you time back to focus on what's important; Patient Care.