




ASSEMBLY GUIDE

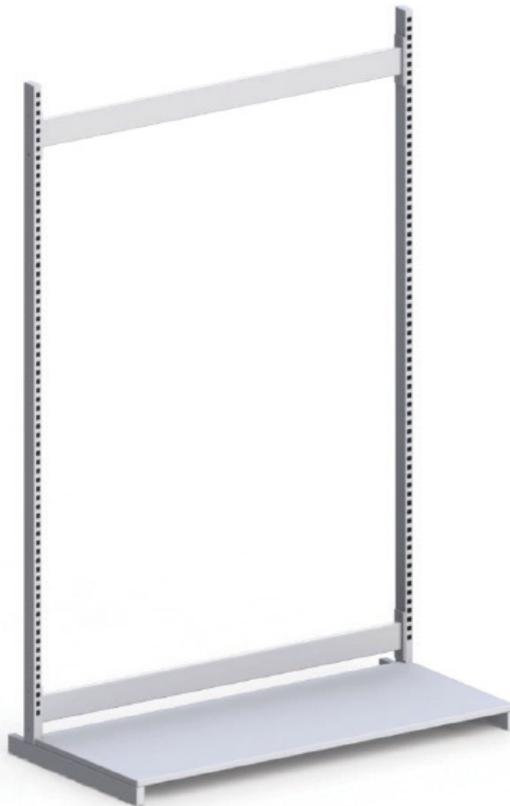

ParWall[®]

FRAME ASSEMBLY



CAUTION


Read and understand all of the instructions and safety information in this document before assembling this product.

PARWALL® FRAME PARTS & HARDWARE


TOOLS NEEDED

- $\frac{9}{16}$ " Wrench
- $\frac{1}{4}$ " Socket and Ratchet
- $\frac{5}{32}$ " Hex Key

PARTS & HARDWARE	QTY
1. Frame	2
2. Frame Cross Bar	2
3. Dust Cover	1
4. Flat Head Screw ($\frac{1}{2}$ -20 x $\frac{3}{4}$ ")	8
5. Sheet Metal Screw (#8 x $\frac{1}{2}$ ")	4 single/8 double
6. Brake Casters	2
7. Swivel Casters	2

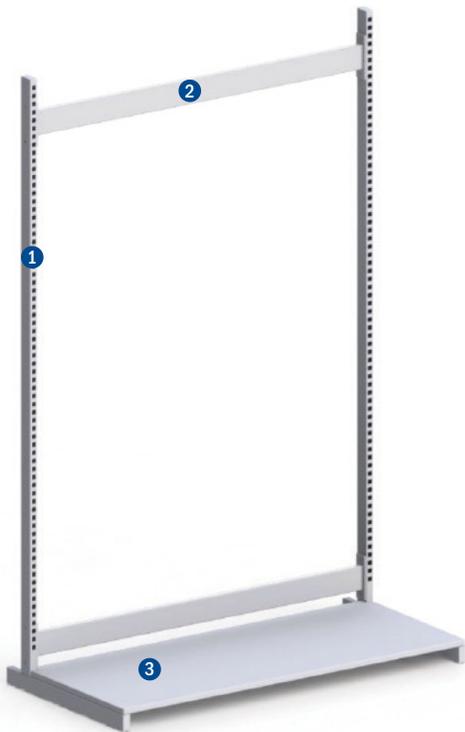

Single Frame

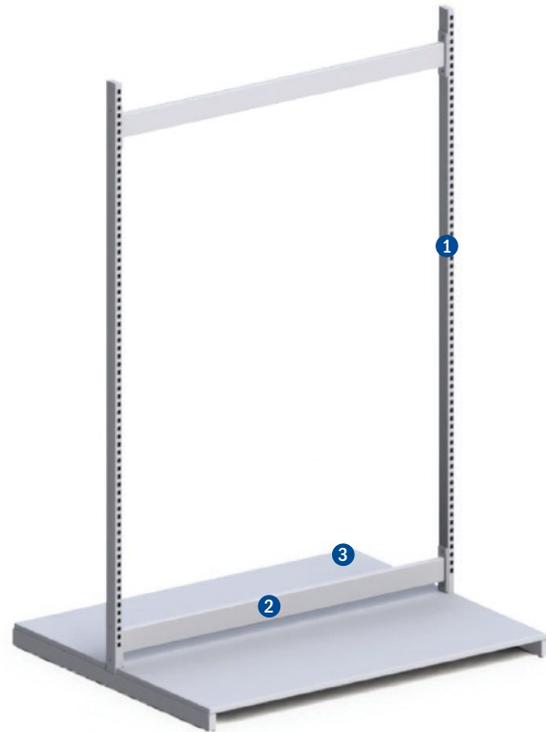


Double Frame



ASSEMBLY INSTRUCTIONS

Assemble the Frame and Cross Bars as shown below using a $\frac{5}{32}$ " Hex Key (hand tighten only) and eight (8) Flat Head screws (PART 4).

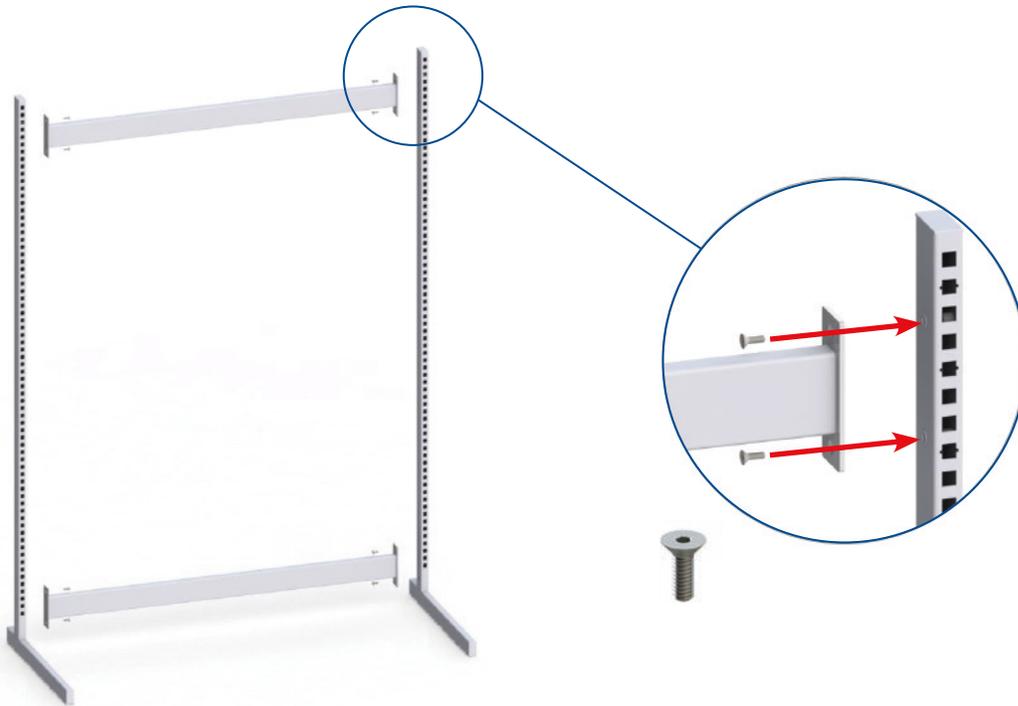

Attach the Dust Cover to the Frame using a $\frac{1}{4}$ " Socket Wrench (hand tighten only) and four (4) Sheet Metal screws (PART 5) for a Single Frame or eight (8) Sheet Metal screws for a Double Frame.



ASSEMBLY INSTRUCTIONS (CONTINUED)

Attach the Casters – 2 Brake (PART 6) and 2 Swivel (PART 7) to the bottom of the Frame. Place the Brake Casters in the front and the Swivel Casters in the back. Thread the stems of the Casters into the threaded holes at the bottom of the bottom of the upright by hand.



Lastly, with your $\frac{9}{16}$ " wrench, tighten down the Casters by turning the Bolt under the Caster Housing clockwise until snug.





LIMITED 5-YEAR WARRANTY

LogiQuip products are covered by a Limited 5-Year Warranty from date of receipt against defect in materials or workmanship under normal use and service, excluding Electronic Components, Cart Covers, Custom Products, LogiCell Tambour Doors, Label Holders, and Casters. During this period, LogiQuip will, in its sole discretion, repair defects in materials or workmanship that existed when the product was received or replace the product. Visit [LogiQuip.com/Warranty](https://www.logiquip.com/Warranty) for more details.

CUSTOMER SERVICE

If you have any problems with your ParWall® Frame, please contact our Customer Service Department at:

PHONE: **(800) 665-3760**

EMAIL: CustomerExperience@LogiQuip.net

ABOUT US

Healthcare exclusive since 1992, LogiQuip's Inventory Management Solutions are designed for Healthcare Professionals looking for a better way to manage supply chain tasks. With Expert Advice, Innovative Solutions, and 360 Service that doesn't stop after the sale, LogiQuip's sole purpose is to ensure Healthcare Professionals have the supplies they need when they need them. Partnering with you to design and implement the right solutions, we give you time back to focus on what's important, Patient Care.